

## WHAT IS THE CRYOCORRECT TREATMENT?

The SkinCeuticals CryoCorrect Treatment is an affordable and comfortable way to gently freeze away unwanted age spots, sun spots, and skin tags from the face, hands, and body.

With little to no downtime and **minimal discomfort**, results can be seen in as little as one treatment. Consult your physician or skincare professional to determine if a CryoCorrect Treatment is right for you.



## COMPLEMENT YOUR TREATMENT BY PAIRING A TRIPLE ANTIOXIDANT SERUM WITH A BROAD-SPECTRUM SUNSCREEN



### PHLORETIN CF®

- Significantly improves skin smoothness of the treated area at one week post-treatment\*
- Improves general appearance of the treated area at one week post-treatment\*
- Diminishes the appearance of discoloration

\*Compared to CO2 alone



### SHEER PHYSICAL UV DEFENSE SPF 50

- Helps protect skin from future discoloration
- Safe for use on sensitive skin
- Ultra-sheer formula

 SKINCEUTICAL

SUN  
SPOTS

SKIN  
TAGS

AGE  
SPOTS

# REMOVE UNWANTED SPOTS

WITH THE CRYOCORRECT TREATMENT

- Ask about this non-invasive, in-office treatment that gently freezes away spots and skin tags on the face, hands, and body in as little as 1 treatment.


EFFECTIVE | SAFE | FAST

## VISIBLE IMPROVEMENT IN DARK SPOTS

Significant improvement was seen among dark spots of varying size and shade, with 66% of subjects showing complete removal of dark spots by week 6.




Photos represent above average results

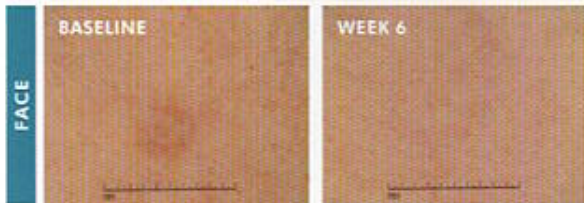

Photos represent average results

## COMPLETE REMOVAL OF SKIN TAGS

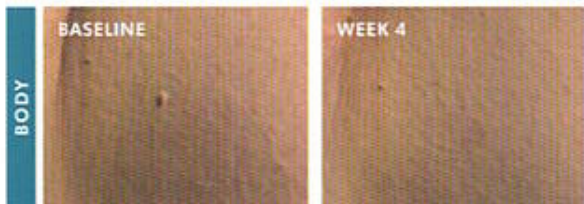


Photos were captured using a 3D imaging system

## HOW IT WORKS

The CryoCorrect Treatment features the SkinCeuticals CryoCorrect PRECISION™ pen which dispenses a precise flow of carbon dioxide (CO<sub>2</sub>).


Carbon dioxide freezes the skin at a milder temperature than traditional cryogens like liquid nitrogen. Carbon dioxide's mild temperature results in a **softer, shallower, more comfortable freeze**, more likely to leave the surrounding skin intact.

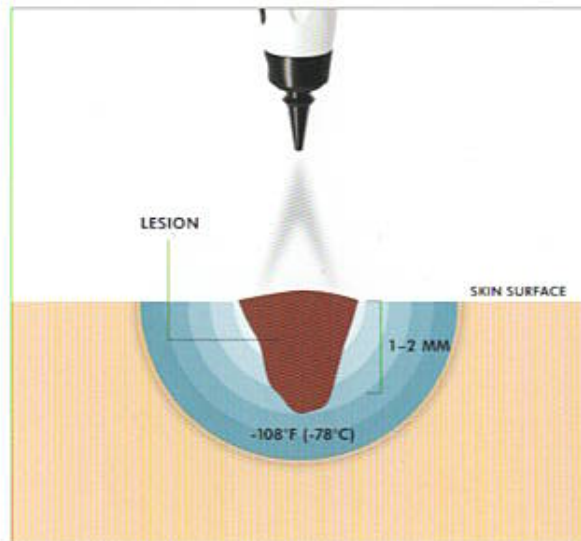

## WHAT TO EXPECT

During your treatment, the spot or skin tag is quickly frozen for 5-8 seconds with a precise stream of carb dioxide (CO<sub>2</sub>) to gently freeze away the unwanted skin cells.

Depending on your response, the recovery process varies from 4 to 12 weeks. Visible improvement can be seen in as early as 4 weeks with healthy skin appearing from week 4-12.

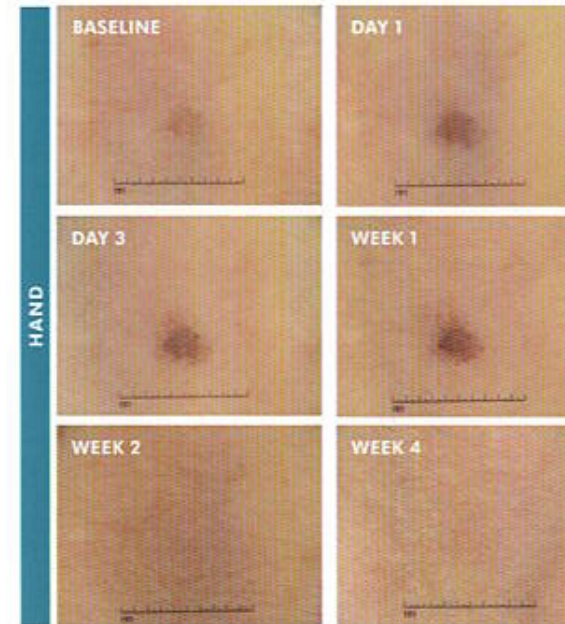

### DARK SPOTS AT 6 WEEKS:

98% of patients showed improvement in dark spot intensity

86% of patients showed improvement in dark spot size

### SKIN TAGS AT 4 WEEKS:

96% of patients saw complete removal of a skin tag